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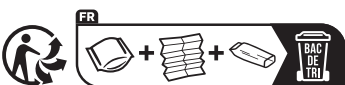
Instructions for use (en)

This condom is free of natural rubber latex.
The female condom, also called the internal condom is designed to be retained in the vagina during sexual intercourse to prevent pregnancy and the transmission of sexually transmitted infections. Store in a well ventilated, cool and dry place, away from sources of heat including direct sunlight. Not to exceed average temperature of 30°C. No barrier method, including male condoms or female condoms, provide 100% protection against pregnancy and STIs. Complies with ISO 25841:2020.

For contraceptive use : When used according to the manufacturer's instructions, condoms are an effective form of contraception.

For the prevention of sexually transmitted diseases : If used correctly, condoms help reduce the risk of transmission of sexually transmitted diseases.

Important things to note for the best protection with the So Sexy & Smile® :

- Before you try the condom, be sure to read the directions and learn how to use it correctly.
- Practice putting the condom in the vagina without having sex to get familiar with how it works.

Using the So Sexy & Smile® :
• The condom is inserted before sex. Most people insert the condom between 2 to 20 minutes before sex.
• The condom can be used with most commercially available sexual lubricants.

Risks of using So Sexy & Smile® :

- Use the condom correctly every time you have sex. If the condom is not used correctly every time, your risk of getting a sexually transmitted infection or becoming pregnant will be higher.
- Incorrect use may result in the condom not providing protection. To avoid risk of incorrect use, read the instructions carefully prior to use.
- The following events are rare but can occur when using the condom :
 - Discomfort during insertion
 - Pain after insertion, before sex
 - Discomfort during sex
 - Burning sensation, rash or itching

Any serious incident that has occurred in relation to the device must be reported to the manufacturer and the competent authority of the Member State in which the user lives.

Before opening :
• Check expiry date on sachet.
• Spread lubrication around by rubbing packet with hands.

Open sachet : tear down from top of sachet.

1. The condom is made of nitrile (synthetic).
Inner ring, used for insertion and holds condom in place.
▲ Outer ring, covers the external genitalia.
▼ Squeeze sides of inner ring together to form a point.

2. Insert squeezed ring into vagina.

3. Push up as far as it will go. Condom should not be twisted.

4. Before sex. Guide penis inside condom.

6. Removal. Hold and twist outer ring, pull out.

7. Discard in rubbish bin. Do not flush. Flushing condom would be an environmental hazard.

Hints :
• Add extra lubricant if :

- Outer ring is pushed inside the vagina.
- Penis sticks.
- You hear noise during sex.

• Be careful of sharp nails and jewellery to avoid tearing the condom.

• Seek medical assistance as soon as possible, at least within 72 hours, should the condom leak or burst during use.

• If the sachet is obviously damaged discard the condom and use a new one from an undamaged sachet.

• Consult a doctor or pharmacist about the compatibility of topical medicines that may come into contact with condom .

• The condom is for single use only and cleaning and reuse can compromise the integrity of the device.

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