

 Distributeur / Vertriebspartner / Distributor

LOT Code de lot - Chargencode - Batch code

 Contenu en latex de caoutchouc naturel -
Gehalt an Naturkautschuklatex
Content in natural rubber latex

 Ne pas réutiliser
Nicht wiederverwenden
Do not re-use

 Attention -Achtung - Warning
 **Identification Unique des Dispositifs**
Eindeutige Identifikation von Produkten
Unique Device Identification

PLEASURE LATEX PRODUCTS SDN.
Lot 5322, 15th Miles, Jalan Padang
Selangor Darul Ehsan-Malaysia.



Ouvrez délicatement l'emballage et avec précaution pour ne pas abîmer le préservatif.


5-Enveloppez le préservatif utilisé dans un mouchoir, jetez le préservatif usagé à la poubelle (pas dans la cuvette des toilettes), après l'avoir noué. Conserver les préservatifs dans un endroit frais et sec, à l'abri de la lumière directe du soleil, de la chaleur, des températures excessives et de l'ozone.

2-Prüfen Sie die richtige Position der Außenseite vor dem Überziehen. Legen Sie das gerollte Kondom über den Peniskopf / die Eichel und lassen Sie am Ende des Kondoms einen freien Platz zum Auffangen der Samenflüssigkeit.

*** Warnhinweis:**

2-Check that the part to be unrolled is outside, Place the rolled condom over the head of the penis and leave an empty space at the tip of the condom to collect the semen. Pinch the tip of the condom with your thumb and index finger, move on to the next step.

* Any serious incident related to the male condom should be reported by the user of this male condom to the manufacturer and the competent authority of the Member State in which the user of this male condom resides.

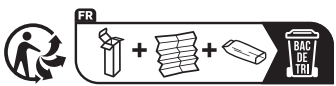

The diagrams show the following steps:

1. Rubbing palms together with fingers interlaced.
2. Rubbing the back of the right hand with the left palm.
3. Rubbing the back of the left hand with the right palm.
4. Rubbing the palms together with fingers spread.
5. Rubbing the palms together with fingers spread, in a different orientation.

 Distributore / Distributeur / Distribuidor

 Scadenza per l'utilizzo
Deadline voor gebruik
Fecha límite de utilización

LOT Codice lotto - Code van het lot - Código del lote

 Conservare lontano dalla luce del sole

Mantener protegido de la luz solar

Contenuto in lattice di gomma naturale -
Natuurlijk rubber latex inhoud

 Contiene latex y caucho natural

 No utilizar si la envoltura está dañada
 Non riutilizzare
 Niet hergebruiken

 Fare riferimento alle istruzioni per l'uso
Raadpleeg de gebruiksaanwijzing

 **Attenzione: - Opgelet - ¡Atención!**

UDI	Unieke apparaatidentificatie Identificación única del dispositivo
MD	Dispositivo medico Medisch apparaat

PLEASURE LATEX PRODUCTS SD

Obelis S.A. Boulevard général Wahis 53

Tel : +(32) 2.732.59.54 - E-mail : mail@obelis.net




Terpan
 Importato e distribuito da
 Importateur en gedistribueerd door
 Importador y distribuidor
 14 rue du Clos Hubert,
 28320 Gallardon, France.
 www.Terpan.fr

dello Stato membro in cui risiede l'utilizzatore.

Todo incidente grave relacionado con el preservativo masculino debe notificarse por el usuario del preservativo masculino en cuestión al fabricante y a la autoridad competente del estado miembro en el que el usuario de este preservativo masculino se encuentre.